


Actichlor Plus

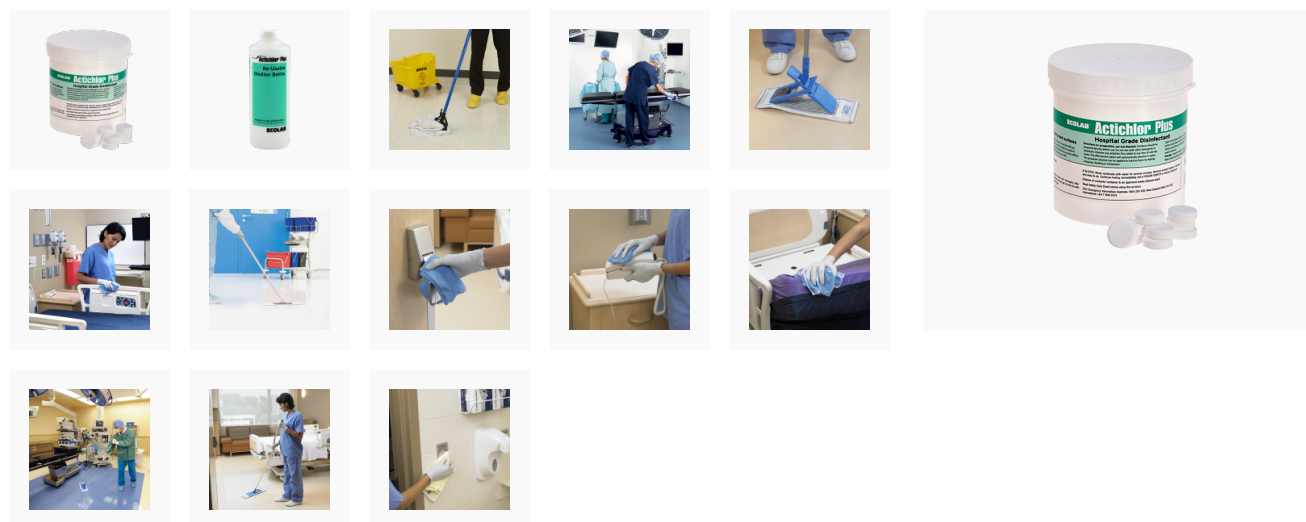


Cleaning & disinfectant effervescent tablets - 150 Tablets


ARTG 335352



Features

- Hospital grade disinfectant and cleaning effervescent tablets.
- Actichlor Plus is a one-step chlorinated detergent and hospital grade disinfectant which cleans and disinfects in a single application.
- Actichlor Plus is suitable to be used in “dirty conditions”.
- New formula effervescent tablets dissolve faster with a lower chlorine smell.
- Actichlor Plus is a broad spectrum bactericide, yeasticide and virucide effective against both enveloped and non-enveloped viruses including polio, Herpes Simplex, Norovirus and Human Coronavirus COVID 19 [SARS-CoV-2].
- Actichlor Plus tablets are designed as a convenient portion controlled system that guarantees a minimum of 1,000ppm chlorine when dissolved in 1 litre.
- The tablet format ensures chlorine stability overcoming the limitations of liquids where chlorine levels degrade over time.
- Actichlor Plus cleans and disinfects in a single step and is suitable for use in public areas, patient rooms, operating theatres and ICU's.
- For patient rooms, Actichlor Plus is suitable for daily cleaning, terminal and infectious room cleans.
- The use of Actichlor Plus at 1,000ppm conforms with NHMRC and ACSQHC Australian Guidelines for the Prevention and Control of Infection in Healthcare settings.

Product Code	Product Description	Pack Size	Quote
7100086	Actichlor Plus	6 x 150 tablets	
7100086S-KA6	Actichlor Plus Bottle	6 Pk	
99001127	Asepti Dry Wipes	8 x 100 wipes	

Product Code	Product Description	Pack Size	Quote
ISG-00062	Procedure & technique wall charts - Actichlor Plus	ea	

Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.

ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

Directions For Use

Single Step Cleaning and Disinfection

- Dissolve 1 Actichlor Plus tablet in 1L of cold water (delivers 1000ppm of available chlorine).
- The effervescent tablet will automatically dissolve in water.
- The prepared solution can be applied by wiping, mopping, flooding or immersion.
- Dissolve 10 Actichlor Plus tablets into 1L of cold water to make a solution of 10,000ppm available chlorine.
- When necessary residues can be removed by rinsing or wiping.
- Not intended for use on medical devices. Discard solution within 24 hours or at the end of the shift.

Related Products

- [Asepti Neutral Wipes, 325 x Pail, 100 x Soft Pk](#)
- [Asepti Active Liquid, 500 mL](#)
- [Asepti Active Wipes, 325 x Pail, 100 x Soft Pk](#)
- [Asept Alcohol Wipes, 80 x Soft Pk](#)
- [Asepti Active 24/7 Liquid, 500 mL](#)
- [Asepti Disinfectant 1 Liquid, 500 mL](#)

Applications

- Theatre
- Dental
- Healthcare Facilities
- ICU
- Patient Room
- Patient/Public Bathrooms
- Waiting Rooms
- Endoscopy
- Residential Aged Care
- CSD
- Day Surgeries
- Wards
- Emergency
- Veterinary

Resources

- SAFETY DATA SHEET (SDS)
- Video

- Brochure