

Asepti NewGen

Non-hazardous non-enzymatic metal safe detergent providing a clean result and staff safety

ARTG 122628



Features

- The non-hazardous classification and non-caustic composition of Asepti NewGen together with its effective cleaning performance make it an excellent choice for ward washer disinfectors.
- Asepti NewGen offers a combined objective of a clean result and staff safety.
- Asepti NewGen is a mild alkaline detergent used for manual and machine cleaning of surgical, laboratory and dental instruments.
- Asepti NewGen is soft metal safe and is suitable for cleaning medical utensils such as surgical steels, plastic, elastomers, anodized and non-anodized aluminium, titanium, anaesthetic masks, circuits and connectors.
- Asepti NewGen is suitable for use in utensil, ward, pantry, batch, ultrasonic, index, cart and trolley medical and glassware washer disinfectors.
- Asepti NewGen complies with AS 5369:2023 and AS/NZS 4815:2014.

Product Code	Product Description	Pack Size	Quote
7100737, 7105473	Asepti NewGen, 5 L	2 Pk	
7101742	Asepti NewGen, 110 L	ea	

Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.
 ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

Directions For Use

Manual Cleaning:

- Recommended dilution is 2 – 8 mL per litre.

Automated Use:

- In utensil and ward washers the recommended dilution is 5 mL per Litre.
- Asepti NewGen is recommended to be used in conjunction with Asepti Rinse Clear or Asepti Citric Rinse in the final high temperature disinfection stage of non-anaesthetic cycles.

Batch Washers, Index Washers, Cage Washers and Trolley Washer Disinfectors:

- In Batch, Index, Cage and Trolley washers the recommended dilution rates are:
- 3 – 8 mL per litre for medical instruments.
- 1 – 3 mL per litre for cage and trolley washers

Relevant Product

- | | |
|-------------------------------|---------------------------------|
| • Smeg GW0260 | • Smeg GW6290 |
| • Smeg GW2145 | • Smeg GW6010 |
| • Smeg GW4060 | • Smeg GW7010 |
| • Smeg GW4190 | • Smeg GW7015 |
| • Smeg GW3060 | • Belimed WD290 |
| • Smeg GW4090 | |

Applications

- Laboratory Infection Control
- Healthcare Facilities
- Central Sterile Departments
- Residential Aged Care
- Life Science

Resources

- safety data sheet (sds)
- Video
- Brochure