

## OptiPro Solid De-scaler

Solid descaler offering a safer way to descale automated washer disinfectors

ARTG 235775



### Features

- OptiPro Solid Descaler eliminates the dangerous use of liquid descalers providing a solid and safer alternative promoting staff safety.
- OptiPro Solid Descaler is portion controlled to offer a safer way to descale your automated washers.
- OptiPro Solid Descaler is formulated to dissolve and remove a variety of residues and stains from washer disinfectors including: lime, scale, uric salts, iron stains, alkaline detergent residues, and other mineral deposits.
- OptiPro Solid Descaler is easy to use and store, helping to eliminate cost, safety and space issues of storing and using liquid descalers. The unit dose allows consistent results and avoids product misuse and waste by providing the same amount of descaler every time.
- OptiPro Solid Descaler is a highly effective periodic and routine descaler for use in batch washers, index washers, tunnel washers and trolley washers as a part of an automated process.
- When used routinely OptiPro Solid Descaler helps to extend the life of your equipment.
- OptiPro Solid Descaler complies with AS/NZS 5369:2023

Product Code	Product Description	Pack Size	Quote
7100720	OptiPro Solid Descaler, 0.45 kg	6 Pk	

### Safety Information

Do not mix with any other products. For further information, safety instructions and health warnings, see product label and Safety Data Sheet.  
 ALWAYS READ THE LABEL. FOLLOW DIRECTIONS OF USE.

### Directions

- Wear suitable PPE. Gloves.
- Remove OptiPro Solid Descaler container cover.
- Turn container upside down and place on a mesh basket
- **Instruments must NOT be included in the descale cycle.** This is to descale the WD only.
- De-scale Program Parameters are:
  - No Pre-wash.

- at the bottom of the wash chamber.
- Push on the bottom of the container to release product.
- Remove the puck from the plastic container.

- Select a 2 or 3 level rack and place puck in a mesh basket on bottom level. Good water flow and mechanical action is essential.
- Wash temperature 65°C - 75°C
- Wash time 15 - 30 minutes.
- Descale cycle MUST be followed by three rinses.

## Relevant Product

---

- [Belimed WD290IQ](#)
- [Belimed WD290](#)
- [Belimed WD200](#)
- [Belimed WD390](#)
- [Belimed WD750](#)
- [Belimed CS750](#)
- [Smeg WD7015](#)
- [Smeg WD7010](#)
- [Smeg WD6010](#)
- [Smeg WD1160](#)
- [Meiko TopClean 60](#)

## Applications

---

- Healthcare Facilities
- Central Sterile Departments
- Residential Aged Care
- Life Science

## Resources

---

- safety data sheet (sds)
- Video
- Brochure